

# Fitting & Care Instructions

**Model: DCT-PT12**

Instructions on putting on brace:



Step 1: Open the brace and detach all the straps.



Step 2: Place your foot through the sleeve and pull up until the circle opening is around your patella



Step 3: Take the two top inner elastic straps and attach to the top square pad



Step 4: Take the two bottom inner elastic straps and attach to the bottom square pad



Step 5: Pull the bottom outer strap through the opposite side D-Ring and tighten to desired support



Step 6: Pull the top outer strap through the opposite side D-Ring and tighten to desired support

**TREND™**  
M E D I C A L

## CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

## WARRANTY

The DCT-PT12 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.