

Fitting & Care Instructions

Model: DCT-QF27



Instructions on putting on brace:



Step 1: Open brace and lay flat on a table



Step 2: On the right and left side lift and detach the overlapping sizing extension



Step 3: Find your size (SM-3XL) and reattach the overlapping sizing system to the correct size



Step 4: Repeat step 3 to the opposite side so both sides are the same size.



Step 5: Once properly sized center the back brace in the middle of your back (logo at the top) and wrap around your waist.



Step 6: Wrap the side wings right side overlapping the left side and pull the pull strings to your desired compression

WARRANTY

The DCT-QF27 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.

Size 0	Size 1	Size 2	Size 3	Size 4	Size 5	Size 6	Size 7
22" - 32"	31" - 37"	36" - 40"	39" - 43"	42" - 46"	45" - 49"	48" - 52"	51" - 55"