

Fitting & Care Instructions

Model DCT-3761



Step 1: Loosen the straps and slide the affected arm into the sleeve



Step 2: Slide the sleeve on the arm until the ROM dial is center at the bend in the patients arm.



Step 3: Locate the ROM dial at the center of the brace and set the dial to the desired flexion and extension



Step 4: Locate the bottom straps and tighten the straps to a secure fit. Note: these straps could be trimmed to length by removing the alligator Velcro clip at the end of the straps



Step 5: Locate the top straps and tighten the straps to a secure fit Note: these straps could be trimmed to length by removing the alligator Velcro clip at the end of the straps



Step 6: Your brace should fit snug and comfortable. You shouldn't be able to move your arm much past the set flexion and extension. If too loose tighten to desired support. Shoulder strap is also included for comfort

Care Instructions and Warranty: 6 Month Warranty against any manufacturing defects.
Hand wash with a light detergent and air dry.